

NUTRICIA FLO CARE® INFINITY™ III PUMP

CLEANING GUIDELINES

IMPORTANT — PLEASE AVOID:

Immersing pumps in water, detergents or other fluids.

Storing the pump in extreme heat or cold (under -40°C up to +50°C).



Pumps may be wiped with a wet cloth and detergent and rinsed under running water.



CLEANING THE PUMP:

Clean and remove any tube feed residue from all sides, the sensor area, and in the corners of the keypad where it meets the pump housing. Use a mild, diluted cleaning solution and damp cloth. Avoid immersing pump. If necessary, use a moist cotton bud to clean sensor area.

DRYING THE PUMP:

Shake pump to remove excess water and allow to air dry for approximately 30 minutes or until completely dry, before plugging in to charge.



NUTRICIA
flocare® Infinity™ III

This information is intended for patients or carers of those who have been prescribed a tube feed by a healthcare professional.